

Instructions for Resetting RESPeRATE (Deleting Performance Statistics)

For units with an orange on/off button:

1. Without turning on RESPeRATE, press the Info button ('i') and hold for 3 seconds. Statistic 1 appears on the display (1:xx).
2. Press the 'Up' arrow button repeatedly until 6 appears on the display (6:xx).
3. Press the Info button ('i') and the Melody button together and then press the "Up" arrow button once while still holding down the first two buttons. ("7:xx" appears on the display.)
4. Press the Session Duration button once and the device is reset and turns off itself.

Please note:

- Although the instructions above seem daunting, the process is really very simple and takes just seconds.
- For RESPeRATE **Duo units**, repeat the process above for each user – User 1 & User 2.
- Resetting RESPeRATE will delete the memory permanently -- important statistics related to the use of the device such as the Weekly Therapeutic Minutes.
- It is generally not required if just one person is using the device, or two people for the Duo device.
- Healthcare providers who are loaning the device to individual patients may choose to reset the device between users.